Varseo XS

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Quick guide

Guide rapide

Breves instrucciones

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This quick guide contains instructions on how to switch the Varseo XS 3D printer on and how to set it up. The complete operating instructions are stored as a PDF file on the provided USB flash drive. Please read the operating instructions carefully before using the 3D printer and pay special attention safety instructions on handling the device.


Scope of delivery

Please keep the original packaging materials. The device must only be shipped in the original packing material as the device may otherwise get damaged during transport. Please check the scope of delivery for completeness:

- Original operating instructions Varseo XS (on the provided USB flash drive)
- Stainless steel spatula
- Power cord (EU and USA)
- Power supply unit
- USB flash drive with activation software
- Container
- Gloves

Device description

- | | |
|--|---|
| <ul style="list-style-type: none"> ① Mains switch ② Build platform ③ Fixing screws for the resin tank ④ Resin tank ⑤ LCD display ⑥ Control panel ⑦ USB socket, type A ⑧ Mains supply ⑨ USB socket, type B | <ul style="list-style-type: none"> ⑩ LAN-port/Ethernet socket ⑪ Place for the resin bottle ⑫ HOME button: Return to main menu ⑬ Left and right button: Navigate through menu items ⑭ Up and down button: Navigate through menu items ⑮ OK button: Confirmation button |
|--|---|

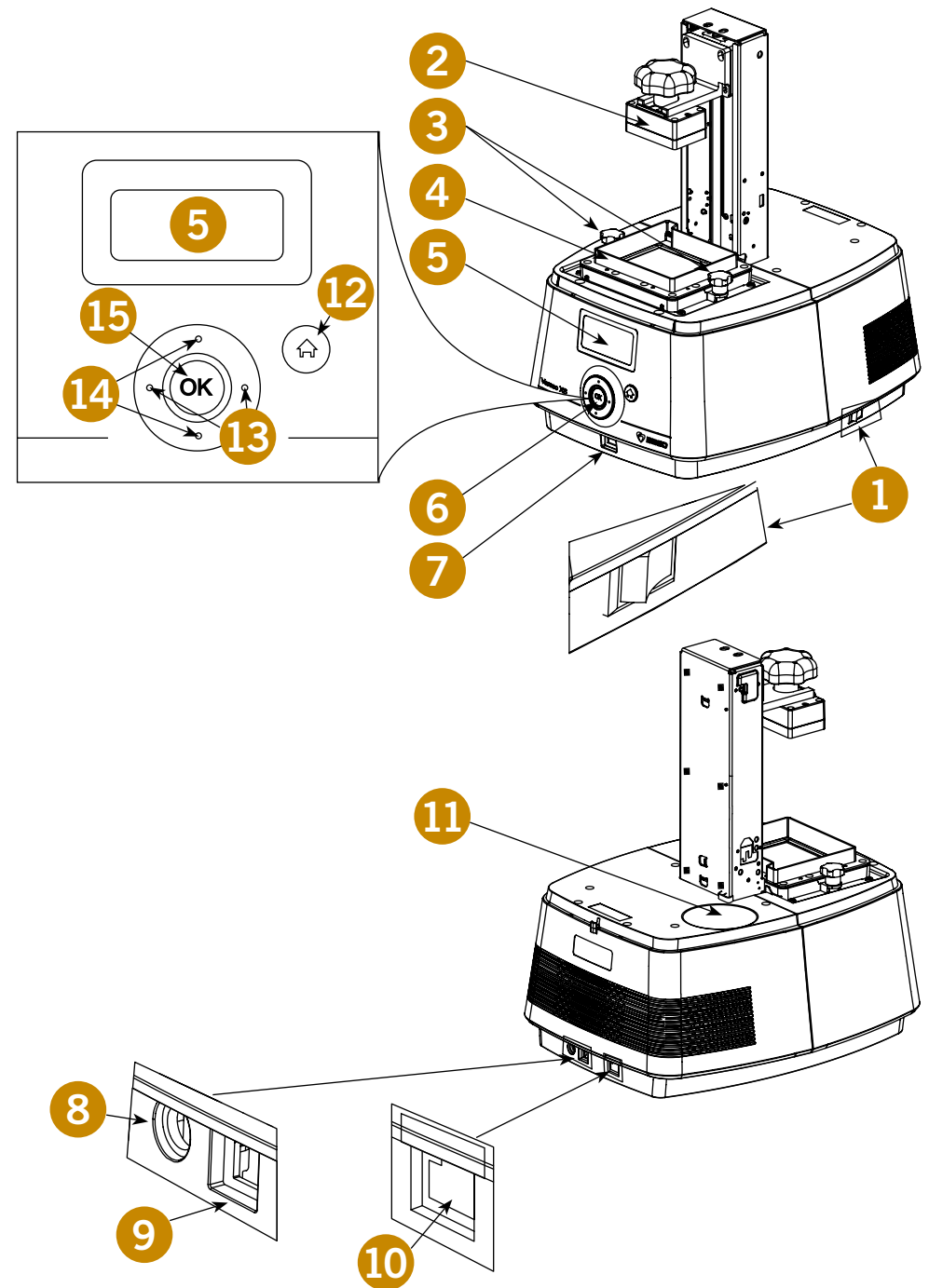
General notes

- Please install and use the printer in a well-ventilated area. The distance to the wall must be at least 10 cm to ensure that there is enough space for proper device ventilation.
- For safety reasons, please put on safety goggles before commissioning the printer.
- Please make sure that the recommended operating temperature of between 20°C and 30°C as well as the relative humidity between 40% and 60% are maintained. If the operating temperature specified on the label of the resin bottle to be used deviates from the operating temperature recommended above, please adhere to the conditions specified on the label.
- Protect the device against frost, direct sunlight and/or ultraviolet radiation. Avoid installing the device close to heat sources (e.g. radiators, ceramic ovens).
- Colour pigments may deposit on the bottom of the resin bottle or resin tank if the printer is not used for more than one week. Shake or stir the resin thoroughly before using it.
- Please observe the corresponding resin's directions for use.
- The resin tank is a wear part. It should be replaced if the bottom of the tank is damaged or hazy to warrant the print quality.
- To warrant the print quality, different resins and/or colours should be used with separate individual resin tanks.

1. Switching on & selecting the language

- Connect the device to a power supply outlet **8** with the supplied power supply unit and the power cord.
- Switch the device on with the power switch **1**. After switching the device on for the first time, the selected language will automatically appear on the display **5**.
- Select the desired display language using the up and down buttons **14**.
- Confirm your selection with the OK button **15**.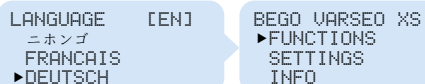


LANGUAGE [EN]
ニホンゴ
FRANCAIS
▶DEUTSCH

BEGO VARSEO XS
▶FUNCTIONS
SETTINGS
INFO

2. Raising the build platform

Upon delivery, the build platform **2** is set to pack mode.

- In order to raise the build platform you must first place the cover on the device.
- Call up the following menu items and confirm every selection with the OK button **15**:

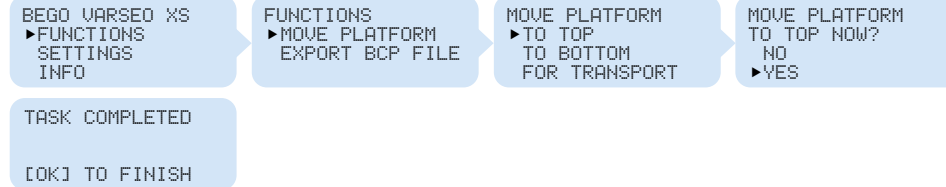- The display indicates the successful completion of the movement.
- Press the HOME-button **12** to call up the main menu.
- Remove the protective material from the resin tank **4** and save it.

3. Establishing a network connection

The Varseo XS printer must always be online to execute print jobs. The network connection can be established via LAN (3a.) or WIFI (3b.).

Both, the PC and the printer, need to be in the same network in order to send print jobs directly from the computer to the printer.

3a. Network connection via LAN


A LAN cable is not included in the scope of delivery.
Use at least a CAT5e patch cable with a RJ45 connector.

The device is set to LAN connection by default. If your network supports DHCP, the device will connect with your network automatically.

- Connect the device by plugging the LAN cable into the ethernet socket **10** on the back of the device and into the "LAN" socket of your router/network connection.

- Check the network connection by viewing the IP address.
- Call up the following menu items and confirm every selection with the OK button 15 for that purpose: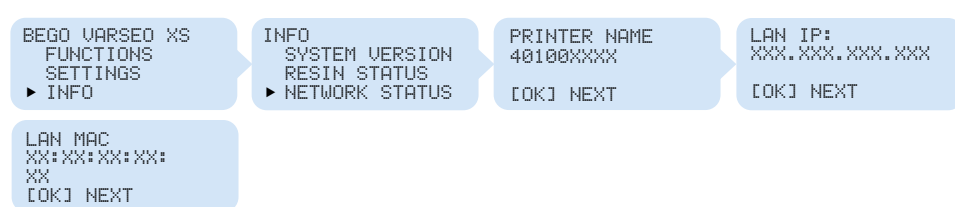


- Press the HOME-button 12 to call up the main menu.
- If an error message appears, check the connectors of the LAN cable on the back of the device 10 and on the router.

 Further information can be found in the operating instructions, chapter: Initial commissioning


3b. Network connection via WLAN

The device is set to LAN connection by default.

- To use the device via WIFI, call up the following menu items and confirm every selection with the OK button 15: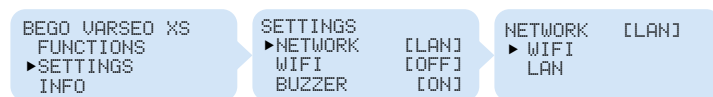

- Connect the printer with a WIFI network.
- Call up the following menu items and confirm every selection with the OK button 15 for that purpose:

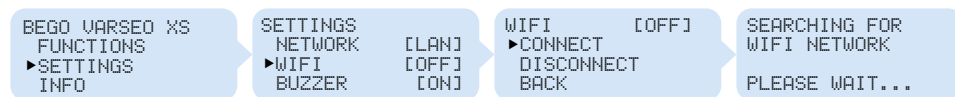The device starts searching for available WIFI networks. Wait until the process is completed. The available WIFI networks are displayed on the device, e.g. (06/20).

- Select the WIFI connection to connect the device with from the list.
- Select the following menu item to enter the WIFI password:


 The key pad and the HOME button 12 must be used together to enter the password.

- Enter the password with the keypad. Four input modes are available, which can be toggled by pressing the HOME button 12:
 - English upper case letters (A–Z),
 - English lower case letters (a–z),
 - Arabic numerals (0–9) and
 - special characters (!, ,, # \$ % & ' () * + , - . / : ; < = > ? @ [¥] ^ _ ' { | } →).


The currently selected entry mode is indicated in square brackets on the top right of the screen. In password entry mode, the keys have following functions:

- HOME-Button 12: Change entry mode
- Up and down buttons 14: Select numerals, letters or special characters
- Left and right buttons 13: Switch to previous or next digit
- The password is shown in clear text when it is entered.
- Press the OK button 15 to complete the password entry.
- After the printer has established a connection with the selected network, complete the process by pressing the OK button 15.

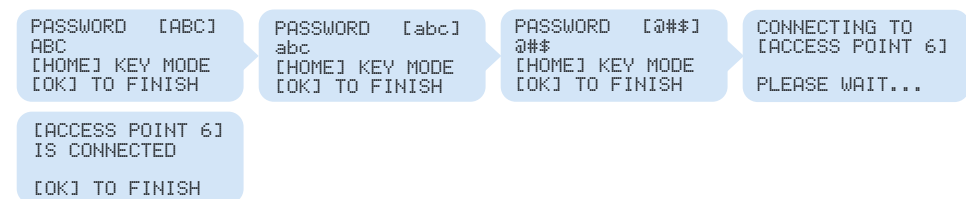- Press the HOME button 12 to call up the main menu.

- Check the network connection by viewing the IP address.
- Call up the following menu items and confirm every selection with the OK button **15** for that purpose:

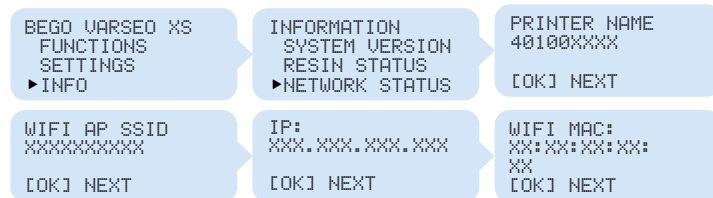- Press the HOME-button **12** to call up the main menu.

 Further information can be found in the operating instructions, chapter: Initial commissioning

4. Exporting machine parameters

Every device has individual machine parameters. These parameters are stored in a bcp file on the device and are required to activate your device on my.BEGO.com as well as for the CAM software BEGO CAMcreator Print. These parameters can be downloaded onto the USB flash drive included with the scope of delivery as follows:

- Plug the provided USB flash drive into the USB port **7** located on the front of the device.
- Call up the following menu items and confirm every selection with the OK button **15**:

- Remove the USB flash drive from the 3D printer.
- Connect the USB flash drive to your computer. The machine parameters are stored on the USB flash drive as a BCP file in the "Profile" folder.
- Copy the exported BCP file to your computer, as needed, to facilitate easy access to the parameters in the future.

5. Registration on my.BEGO.com

In order to activate your BEGO Varseo XS 3D printer, the printer must first be registered on my.BEGO.com.

- Export the machine parameters (bcp file) from the device to the provided USB flash drive as described above.
- Open the "BEGO Varseo Activation.exe" file in the "Profile" folder on the provided USB flash drive.

- Enter all required data and confirm your entries to activate your printer with my.BEGO.com.

The activation will take place automatically.

After registration, an e-mail containing the user information that you have chosen will be sent to the e-mail address you have specified.

- Download the BEGO CAMcreator Print software from your my.BEGO.com account and install it on your computer. Instructions on how to import the machine parameters (bcp file) to the BEGO CAMcreator Print software can be found in the software's installation instructions. These can also be accessed on my.BEGO.com.

After the printer has been successfully activated on my.BEGO.com and the BEGO CAMcreator Print software has been installed, print jobs can be generated and objects can be printed with the 3D printer.

Please note that the Varseo XS printer must be registered on my.BEGO.com and online to execute print jobs!

Further information can be found in the operating instructions, chapter: Initial commissioning

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